MADE IN ITALY

EMIE[®]
ITALY

● ● ● **LÁSER DE ALTA POTENCIA**
HIGH POWER LASER

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Founded in 1983 in Pesaro, EME has established itself over the years as a leading company in the production of electromedical products for physiotherapy, aesthetic medicine and aesthetics. All EME technologies are the result of continuous research and a strong synergy between the internal Research and Development department, the real heart of the company, and specialists in the sector and institutions.

This allows us to offer more performing and innovative solutions and to update the existing ones thus offering the market 100% Made in Italy, cutting-edge and highly reliable products. Currently our products are distributed in over 60 countries worldwide and are globally recognized for quality and performances, so much so they have been chosen by sports teams, sports federations and high level clinics.

*Anyway what matters more than numbers and references is the vision that has leaded us and inspired us for more than 30 years: **“Offer the opportunity for everyone to feel good with their body, in health and harmony”.***

Every day we work for this: to guarantee concrete results for patients all over the world. We do this offering our partners cutting-edge technologies and specific training to achieve the best possible results.

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*The EME High Power Lasers meet MDD 93/42/EEC medical directive and are certified **CE 0476**. EME is a company certified **ISO 9001: 2015, ISO 13485: 2016** and **100% Made in Italy**.*

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One of the strongest foil fencer ever, Olympic gold medalist in London 2012 and silver medalist in Rio 2016.

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Fast and effective ●●●

*The high power laser is a very appreciated technology thanks to the excellent results that could be reached in a very short application time. The EME high power lasers of EME, thanks to the **high power** (from 8 W to 25 W), **real and guaranteed** by using a single diode, allow a high and deep energetic transfer, and consequently a substantial reduction in time of treatment.*

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The versatility of 3 wavelengths

*The EME high power lasers **cover the physiotherapeutic window** and interact with different biological tissues.*

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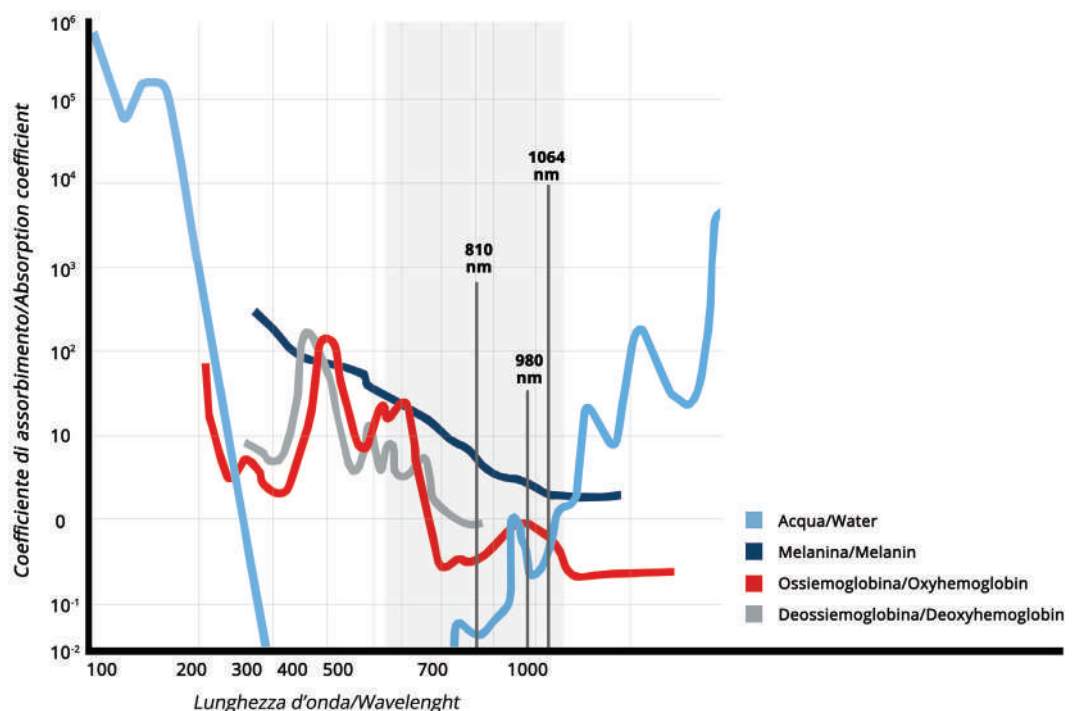
The high power lasers are based on the principles of LLLT (Low Level Laser Therapy) but using more energy, pain is blocked immediately.

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Photomechanical action

The high laser energy impacts on the tissue surface generating real pressure pulses that reactivate the microcirculation and stimulate the deep lymphatic system. This leads to a consequent oxygenation of the tissues and reabsorption of the liquid accumulations.

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Photochemical action

The energy absorbed by the chromophores in the tissues induces chemical modifications: structural modifications of the molecules and reactions that facilitate the enzymatic activation and the synthesis of nucleic acids and proteins.

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Photothermal action

The electromagnetic energy is transformed into thermal energy inducing a controlled increase in the tissues temperature, with consequent stimulation of the circulation and increase in the supply of oxygen and nutrients to the suffering areas.

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Biostimulating and regenerative effect

It facilitates cellular energy processes, growth and tissue repair. Stimulates the production of collagen and elastin and the production of ATP.

Analgesic effect

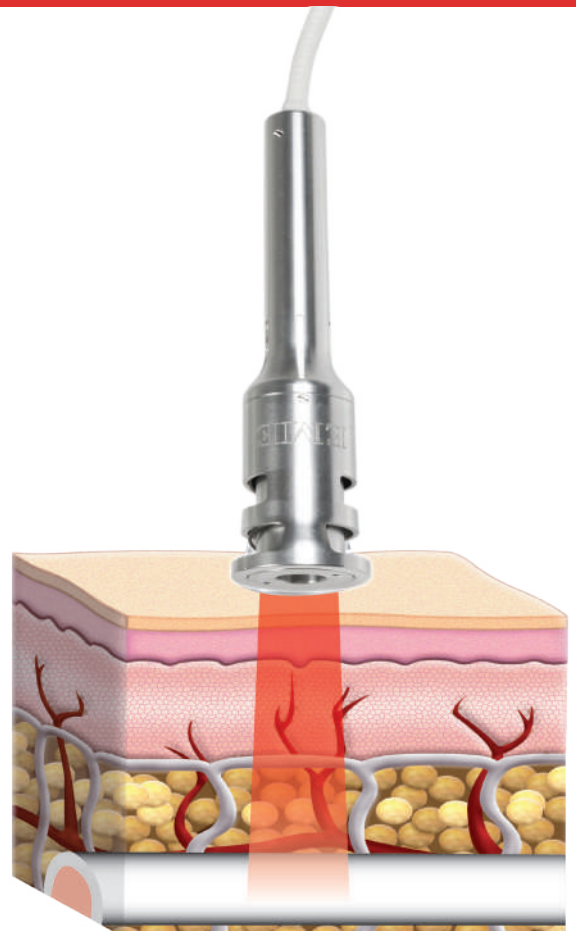
It stimulates the peripheral nervous system and the nociceptors in the subcutaneous tissue, blocking pain and giving immediate relief (Gate Control Theory).

Anti-inflammatory and anti-edema effect

It accelerates the resolution of inflammation by increasing vasodilation with consequent tissue oxygenation and activation of metabolic processes.


Effect on the microcirculation

It has an intense vasoactive action on the microcirculation and lymphatic drainage. The activation of the microcirculation facilitates the supply of oxygen, nutrients and drainage of catabolites from the tissues.

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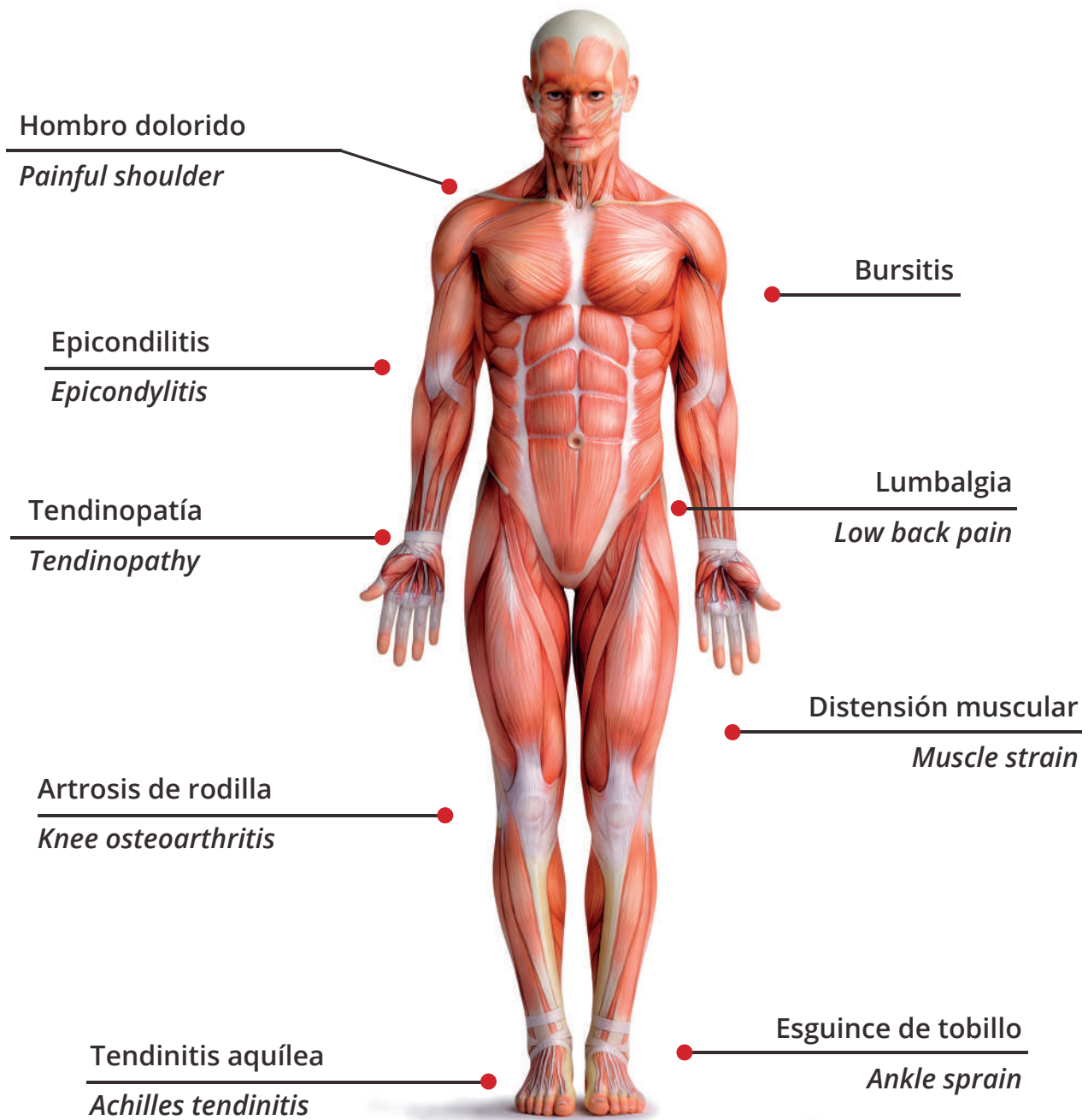■ **Protocols divided by anatomical area**

The easy to use and fast software allows to choose **up to 66 different protocols**, divided by **anatomical area**. Additional 200 customized protocols can be stored.

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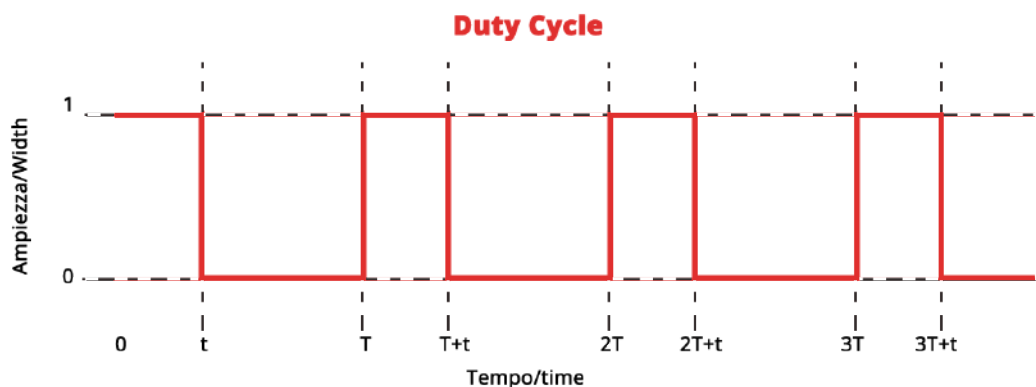
● **Continuous mode and pulsed mode**

The EME high power lasers can work both in continuous mode or in "Duty Cycle" pulsed mode. The Duty Cycle allows to decrease the laser photothermal effect (which determines an increase in the local temperature). Treated tissues will cool rapidly, between one pulse and other, making more effective the biostimulation effect.

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● **4 differents spots in one probe** thanks to the practical spacing system.

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Technical features ●●●

Features	Bipower Lux 12 W	Crystal Yag Desk	Vikare 8 W
Wavelength	810 + 980 nm	1064 nm	980 nm
Peak power in continuous mode	12 W	12 W	8 W
Source type	diode	diode	diode
Duty Cycle	10 - 100%	10 - 100%	10 - 100%
Emission	continuous and pulsed	continuous and pulsed	continuous and pulsed
Emission frequency	100 - 10000 Hz	100 - 10000 Hz	100 - 10000 Hz
Adjustable treatment time	up to 99 minutes	up to 99 minutes	up to 99 minutes
Stored protocols	34 man + 34 woman	33 man + 33 woman	32
Storable protocols	200 + USB	200 + USB	200 + USB
Display	colour touch screen 8"	colour touch screen 8"	colour touch screen 8"
Dimensions - Weight	39 X 17,5 X 28 cm - 7,5 kg	39 x 17,5 x 28 cm - 4,65 Kg	39 X 17,5 X 28 cm - 5 kg

Supplied accessories	Bipower Lux 12 W	Crystal Yag Desk	Vikare 8 W
Defocused laser proben with optical fibre, integrated with the equipment	1	1	1
Defocused laser probe with optical fibre, 4 spot sizes detachable from the equipment	-	-	-
Laser protective goggles	2	2	2
Pedal	1	1	1
Interlock	1	1	1

Via degli Abeti 88/1, Pesaro (PU) 61122 | T +39 0721.400791 F +39 0721.26385
info@eme-srl.com | www.eme-srl.com

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© Getty Images